

Nothing stands in the way of a Wildebeest!







gives monster truck stability.

The hourglass frame is rigid for excellent

energy transfer and the long wheelbase

Accessories

Headrest has vertical, forward and backward, and pivot adjustments. Attaches with knobs to the push handles on the back canes. Includes a ledge for the occipital lobe of the user's head.





These lateral trunk supports bolt on to the back canes. Each pad is adjustable for depth, angle, height and width. The pads have a 4 x 5 array of holes on the back. Each pad includes a washable fabric cover.

The activity tray supports a range of activities like fine motor exercises, eating, and education. The design includes two indentions that are great for objects like a cup, a perimeter rim prevents objects from rolling off.



The polypropylene material is food safe and super easy to clean.



Bolt on anti tipper. Adjustable and flip up.



Tubular armrests with quick release attachments to the frame.




Footrest Riser. Bracket and foot plate that lifts the footrest surface by 80 mm.



Wildebeest Measurements

| ASSISTIVE PRODUCTS | | | | | | |
|---------------------------------|---|---------------|---------------|---------|---------|---------|
| ASSISTIVE PRODUCTS | mm (or as noted) | | | | | |
| Seat Width | 250 | 300 | 350 | 400 | 450 | 500 |
| Seat Depth | 250, 300, 350 | 300, 350, 400 | 350, 400, 450 | | | |
| Back Height | 300, 350, or 400. The Tall Back Kit: 400, 450, and 500. | | | | | |
| Back Angle | 7 angle settings from 78 to 126 degrees at 6-degree intervals. | | | | | |
| Front Seat Height | 470, 490, 510 | | | | | |
| Seat Angle from the Ground | 7 degrees | | | | | |
| Footrest Height | 280 - 380. Footrest Riser Kit: 140 - 300. | | | | | |
| Wheelbase | 530 | 580 | 630 | | | |
| Overall Width | 490 | 540 | 590 | 640 | 690 | 740 |
| Overall Length | 985 | 1,035 | 1,080 | | | |
| Product Weight / Net Weight | 12.1 kg | 12.8 kg | 13.5 kg | 14.3 kg | 15.7 kg | 16.3 kg |
| Weight Capacity | 130 kg | | | | | |
| Center of Gravity Adjustability | Adjustable at 7 settings each 12 mm apart. | | | | | |
| Overall Height | 770 mm (Measured with Back Height set at 300 mm, Front seat set at 470 mm) | | | | | |
| Push Handle Height | 750 mm (Back height set to 300 mm, Front seat set at 470 mm) | | | | | |
| Rear Wheel Dimensions | 600 x 38 mm diameter (24 x 1-3/8 in. 32-540 tire) | | | | | |
| Rear Wheel Camber Angle | 4 degrees | | | | | |
| Caster Dimensions | 200 mm diameter and 70 mm wide natural rubber tire | | | | | |
| Packaging | One unit per carton; partially assembled; 10 minutes assembly time required. | | | | | |
| Package Size | 870 x 650 x 500 | | | | | |
| Package Weight / Gross Weight | 14 kg | 15 kg | 16 kg | 16 kg | 18 kg | 18 kg |
| Certifications and Testing | ISO 9001, ISO 13485, CE, Tested ISO 7176 complete, 2X on double drum durability test - 400,000 cycles, ISWP Caster Test - Pass, Conveyor belt track test calibrated to real world | | | | | |

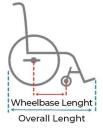



conditions - Pass. (see website for all tests and test reports)




Seat Height

Fit Kit

Items included:

- 1 piece hard plastic sheet 15 x 12 x 1/4" HDPE hard plastic sheet. This can be used to make a seat extension, a mono footplate a wide headrest, to adapt laterals or for other purposes. The plastic is heat gun friendly, is easy to cut with a razor or hack saw, and can be sanded.
- 2 pieces firm foam sheet 15 x 12 x 1/4" EVA foam. Can be cut and stacked up to make pre-ischial, sacral pad, or other modification.
- 2 tubes of rubber cement Can be used to permanently glue EVA, HDPE, etc.

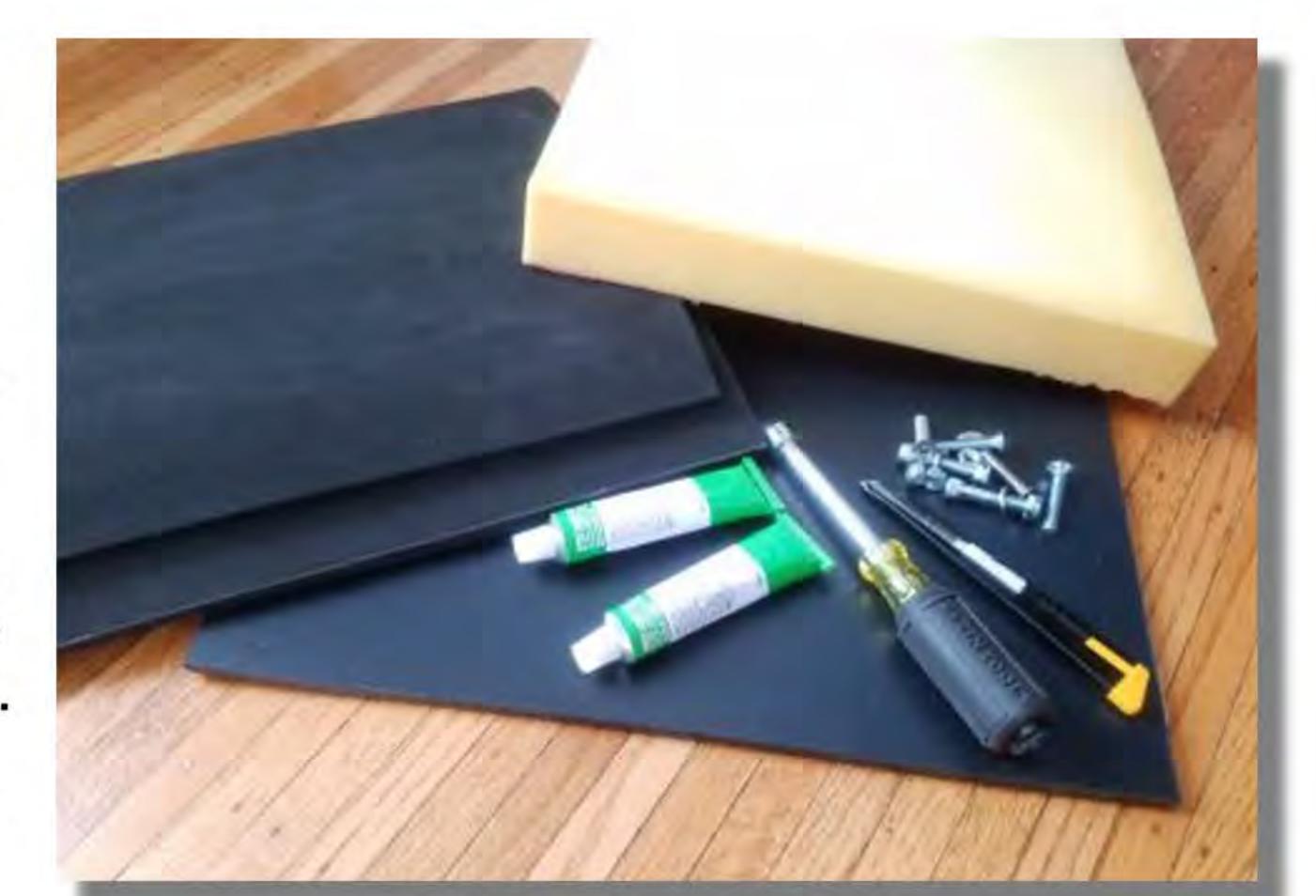

- 1 piece of firm foam 15" x 16" x 2". Polyurethane foam. The density is firmer than the seat cushion. Can be used to make a lift for the seat cushion.
- 1 firm foam wedge, 2" Height x 8" Deep x 15" Wide,
- Hardware M6 screws, nuts, washers.
- Razor knife.
- 10mm nut driver.





